

New Solo HD

Part no: 20120010

User manual



New SOLO user manual version V150216-2

Contents

- Included parts.....page 3
- System requirements.....page 3
- Assembly.....page 4
- Connection to monitor.....page 4
- Remote, User Menu and Service Menu.....page 5
- Connection and installation to computerpage 7
- Short command PC.....page 8
- Technical data, Maintenance & Contact.....page 9

*Congratulation to your choice of visual aid. We hope that
The New Solo HD will enrich your everyday life.*

Included parts

- Camera system
- Table clamp
- Handcontrol
- Powersupply
- USB-cabel
- Monitor cabel HDMI /DVI
- Software CD for computer installation
- Carrying bag

Extra option:

Plate for table

X/Y tablein different sizes

*See www.nordiceye.eu for more information about the
options or contact your local dealer*

Requirements

DVI /HDMI Monitor

PC with USB 2.0/3.0. Win 7 or Win 8

Assembly instructions

Attached the table clamp to the table



Attached the SOLO onto the clamp



Connection to monitor

Connect the DVI/HDMI cable to the SOLO and the monitor
If monitor are used, power supply need to be connected.

Turn on the monitor and the SOLO by the ON/OFF button

Remote



- Z+ zoom in (increase magnification)
- Z- zoom out (decrease magnification)
- Arrow left increase brightness
- Arrow right increase brightness
- FUNC turns picture 180°
MODE changes the picture in three different modes, high contrast positive, high contrast inverted and normal full color mode.

To pairing the remotes and the SOLO after changes batteries or if the units have been swapped:

Power on the SOLO and press and hold down one of the remotes buttons for at least 4 seconds, at the same time you hold the remote as close to the SOLO's plastic body cover as possible.

A red LED will lit up and flashing on the remotes keypad during the synchronization process if the SOLO and remote is not paired.

Release the button and see if there is some response from the SOLO when pressing down a button again.

The remote buttons has multi functions.
Menus and alternative will be available when hold down the MODE button

In the User Menu you can adjust:

- Mirror function
- Line and Curtain
- Colors

In the Service Menu you reach following:

- Display – Brightness, Contrast, Color
- Camera – White Balance, Tilt Function, Display resolution
- Settings – Language, Factory Default

Short cuts on remote

Line and Curtain

- Press the FUNC button for 3 sec and a reading line or Curtain will appear in the live picture

Connection to computer and software installation

Install the SOLO computer software from the attached CD or the downloaded files from www.nordiceye.eu and follow the instructions

Connect the enclosed USB cable to the SOLO and then put in to a free USB port on the computer.

Press ON/OFF button to start the SOLO, Windows will now detect the camera.

Start the SOLO software.

Now you are ready to use the SOLO with your computer by short commands. You can now save pictures and video and store them on your computer.

When using the short USB cable no power supply are needed. If you use the long USB cable use also the powersupply.

Short command PC

Key combination	Function
CTRL-WIN-S	Windows position
CTRL-WIN -↑	Zoom Increase magnification
CTRL-WIN-↓	Zoom decrease magnification
CTRL-WIN-I	Save picture
CTRL-WIN-C	Increase brightness
CTRL-WIN-V	Decrease brightness
CTRL-WIN-T	Turn picture 180°
CTRL-WIN-E	End program
CTRL-WIN-M	Maximize/ minimize program
CTRL-WIN-L	Splitt picture left/right
CTRL-WIN-Y	Splitt picture up/down
CTRL-WIN-Z	Freeze-/release picture
CTRL-WIN-N	Version information
CTRL-WIN-J	Save picture files as.....
Ctrl-WIN-O	Save movie

Technical data

Magnification	2,2 – >85x On 22" Monitor
Focus	Autofocus
Working distance	280 mm
Resolution monitor	1920 x 1080 (Full HD)
Resolution PC	1280 x 720 (HD)
Frame rate	60 full digital frames / second
Connections	USB 2 and 3 And HDMI
Power consumption	18 W
Power	USB or 12 V
Weight	0,95 kg
Operating System	Win 7 & Win 8

Maintenance

Cleaning, do not use any cleaning methods that can result in moisture or water at the electronics.

Deal with the equipment as standard home electronics.

When cleaning the lens be careful so no scratches will appear.

SOLO should be recycled as electronic parts and not be placed in the garbage. Contact your reseller or community to have the product recycled in a non-polluting way.

Contact

If you have any further questions or requests please don't hesitate to contact us.

Or if you only want to leave comments, we are pleased to hear your opinion regarding our products

Nordiceye AB
Illervägen 15
352 45 Växjö
Sverige

Tel +46 470 72 98 88
Fax + 46 470 74 95 80
info@nordiceye.eu
www.nordiceye.eu
